EN *blink*[®] intensive TRIPLE ACTION 3in1 eye drops
soothing • moisturising • protecting
blink[®] intensive TRIPLE ACTION eye drops are formulated to soothe, moisturise and protect dry-feeling or irritated eyes. Combining liposomes, sodium hyaluronate and vitamin E these TRIPLE ACTION eye drops supplement and stabilize the tear film.

Indications
For relief from dry-feeling, sore, watery or irritated eyes.
Suitable for use with contact lenses.

Instructions for use:
Always wash and dry your hands thoroughly before using *blink*[®] intensive TRIPLE ACTION eye drops.

- **Shake before use**
- Instill 1 or 2 drops of *blink*[®] intensive TRIPLE ACTION eye drops in each eye
- Blink to spread over the eye
- Use as often as required
- If you wear contact lenses there is no need to remove them first

Symbols

Consult instructions for use	Manufacturer	Batch code	Use by	25°C 5°C Temperature limitation
Sterile. Aseptic technique	Do not use if package is damaged	Use within 90 days of first opening the bottle		

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Warnings:

- In case of eye irritation, stop using the product and contact a physician
- Do not use if you are allergic to any of the product components.
- Do not touch the eyes with the tip of the bottle
- Keep away from heat sources, store in a dry place between 5 and 25°C
- Keep out of the reach of children
- **Do not use if tamper-evident seal is broken or missing**
- Do not ingest
- Do not use the product after the expiry date
- Use within 90 days of first opening
- Sterile until first opening

• Close the bottle tightly immediately after use

Contraindications: None known.

Composition: A lax coloured, sterile isotonic, buffered solution pH 7.2 containing liposomes, sodium hyaluronate, vitamin E polyethylene glycol succinate (TPGS), polyethylene glycol 400 (PEG 400), disodium edetate, polyhexanide methylbiguanide and purified water. Does not contain chlorhexidine, thimerosal, benzalkonium chloride, phenoxethanol.

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© Johnson & Johnson Surgical Vision, Inc. 2017
blink is a trademark of Johnson & Johnson Surgical Vision, Inc.